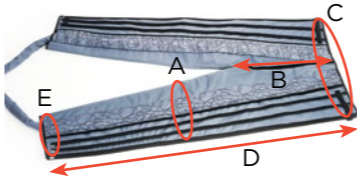
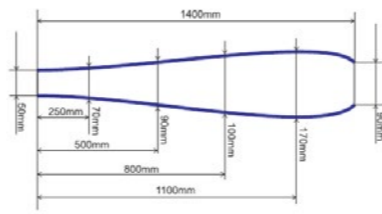
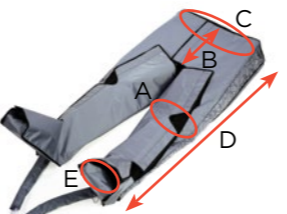
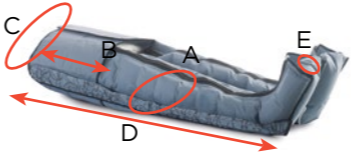
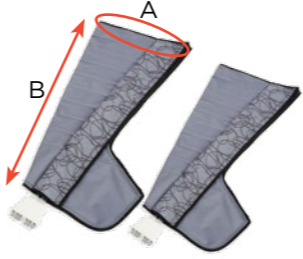
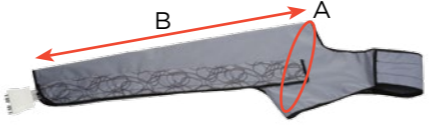



LYMPHASTIM APPLICATORS SPECIFICATIONS

ORDERING NUMBER	NAME	DIMENSIONS	PICTURE	ADVANTAGE	DISADVANTAGE
P6000.060 P6000.061	Trousers 24 chambers Trousers 12 chambers	A: min/max perimeter of the thigh: 66-76 cm B: length of the crotch: 47 cm C: min/max perimeter of the belt: 100-132 cm D: total length: 146 cm E: min/max perimeter of the ankle: 44-54 cm		Unisize trouser applicator - fit patients of different height and weight. Recommended for frequent use.	Some patients don't like this shape, because the ankle part is not as ergonomic as other lymphastim trousers.
P6000.068 P6000.069 P6000.070	Extension strip for leg applicators - size M, S (P6000.066, P6000.062, P6000.076, P6000.077) Extension strip for leg applicators - size L (P6000.067, P6000.063) Extension strip for trousers applicators: (P6000.060, P6000.061)			Enlarged applicator volume up to +17 cm.	The extension strip is not inflated.
P6000.059	Trousers 24 chambers with Velcro	A: min/max perimeter of the thigh: 70-84 cm B: length of the crotch: 37 cm C: min/max perimeter of the belt: 90-144 cm D: total length: 111 cm E: max. volume of the ankle part: 50 cm		The most ergonomic lymphastim trousers on the market. Velcro enables adapting this applicator to patients with different body volume - you don't need any extension strips.	Even high-quality Velcro gets worn off after some time. Handling Velcro is more complicated comparing to zipper. When used for very big patients or with very high pressure, the Velcro could start opening.
P6000.071	Trousers 24 chambers with Velcro - XL size	A: min/max perimeter of the thigh: 80-90 cm B: length of the crotch: 37 cm C: min/max perimeter of the belt: 110-150 cm D: total length: 111 cm E: max. volume of the ankle part: 50 cm		For extra big patients.	Doesn't fit very slim patients.
P6000.072	Trousers 24 chambers with zipper	A: min/max perimeter of the thigh: 80-90 cm B: length of the crotch: 37 cm C: min/max perimeter of the belt: 110-150 cm D: total length: 111 cm E: max. volume of the ankle part: 50 cm		Ergonomic and easy to use. Great combination of ergonomic shape and zipper.	Not flexible in volume size - for bigger patient you need to use extension strip.
P6000.067 P6000.063 P6000.066 P6000.062 P6000.076 P6000.077	Leg 10 chambers - size L Leg 6 chambers - size L Leg 10 chambers - size M Leg 6 chambers - size M Leg 10 chambers - size S Leg 6 chambers - size S	size L: A: perimeter of the thigh: 76 cm B: total length: 94 cm size M: A: perimeter of the thigh: 70 cm B: total length: 78 cm size S: A: perimeter of the thigh: 63 cm B: total length: 78 cm		Ergonomic and easy to use. Great combination of ergonomic shape and zipper.	Not flexible in volume size - for bigger patient you need to use extension strip.
P6000.065 P6000.064 P6000.075 P6000.074	Arm 8 chambers Arm 6 chambers Arm 8 chambers - size S Arm 6 chambers - size S	size L: A: perimeter of the underarm: 45-50 cm B: total length: 100 cm size S: A: perimeter of the underarm: 45-50 cm B: total length: 88 cm		Large breast chamber and Velcro fastening - flexible as to sizes. It's not necessary to open the Velcro very often.	
P6000.034	Interface for 2 arms or 2 legs			Enables using 2 arm applicators or 2 leg applicators together.	